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Teste COVID-19 com controlo humano/ Covid-19 with human control

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Description

The COVID-19 with human control test (ref. 133001001/006) consists of two types of reagents:

1. Tubes with reagents that allow specific amplification of the SARS-CoV-2 virus genes *N* and *E* in a real-time LAMP reaction by fluorescence measurement.
2. Tubes with reagents that allow specific amplification of the human *ACTB* gene in a real-time LAMP reaction by fluorescence measurement (identified with letter *C* on the tube lid).

The COVID-19 with human control test (ref. 133001001/006) is analyzed using two Doctor Vida pocket equipment (ref. 133001002) in parallel and the Doctor Vida mobile phone application (ref. 133001003).

The final result of the COVID-19 test with human control is given by the following matrix:

COVID-19 test	Human Control	Final result
Detected	Detected	Detected
Detected	Not detected	Invalid
Not detected	Detected	Not detected
Not detected	Not detected	Invalid

Indications for Use

The COVID-19 with Human Control test is compatible with nasopharyngeal sample collected in Viral Preservation Medium or pure RNA. The system should only be used by healthcare professionals. The user is responsible for the analysis of the sample and compliance with biosafety procedures, including the use of necessary personal protective equipment, maintenance of facilities and waste management.

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If you require more information, please contact our technical support by email info@doctorvida.com or phone (+351)210438606.